

Restore an array of simple to complex, single or multi-shaded restorations with ease and confidence



Direct Cosmetic Veneer

Restoration of Class IV (Anterior Fracture) with Flowable Frame Technique



1. Bevel enamel at the fracture margin of tooth #11 with Diamond Point 102R.
2. Etch enamel surface, rinse and dry.
3. Apply BeautiBond uniformly on restoration surface.



4. Create the frame by injecting Beautifil Flow F10 (A3T) onto the plastic matrix template.
5. Flowable frame is ready. Build up dentin with Beautifil II (A2) and enamel with Beautifil II (INC).
6. Finish and polish the labial surface with OneGloss.



7. Finish and polish the palatal surface with OneGloss.
8. Finally polish to an enamel-like gloss with Super-Snap fine and Superfine disks.

More restorative options available from the Beautifil Series:

Universal Shades (Beautifil II) – A1, A2, A3, A3.5, A4, B1, B2, C2, Opacious Dentin Shades (A20, A30), Enamel Shade (INC, HVT, LVT, T, AM), Bleached Shade (BW), BEAUTIFIL II GINGIVA (G-Light P, G-Dark P, G-Orange, G-Brown, G-Violet)

Beautifil Opaquer – U0, L0

Beautifil Flow F02 – A1, A2, A3, A3.5, A4, A30, GUM

Beautifil Flow F10 – A1, A2, A3, A3.5, A4, A30, A3T

Beautifil Flow Plus F03 – A0.5, A1, A2, A3, A3.5, A4, A20, A30, INC, BW, MI, CV

Beautifil Injectable – A0.5, A1, A2, A3, A3.5, A4, A0.50, A10, A20, INC, BW

DISCOVER



Bioesthetic Restorations

The Right Tools

Better

Quicker

Easier